

## What is the 2ml IPED Kit (IM)?

This 2ml kit has been developed to provide equipment of the highest quality that complies with N.I.C.E. PH52 Guidelines. It is designed for steroid injectors to promote the best and safest injecting practices. All equipment contained is sterile and for single use only. This pack has been designed for steroid injectors for intramuscular use only.

### Contents:

#### 1 x SAFELOC™0.35 needle exchange container

This is for the safe disposal of used injecting equipment. It fits neatly into a pocket or bag and can display bespoke messages on the label. This container is Kite Mark compliant with BS EN ISO 23907:2012 and Compliant to UN3291 - UN certificate UN/3H1/Y1/S/15/GB/6873. This container is the latest innovative product designed specifically for (IDUs) by Orion.

#### 10 x Acucan Blue 23g 1 1/4" Needle

The needle has three bevels of sharpness and is silicone coated. This provides minimum penetration force and skin trauma during insertion. Additionally, the needle has a thin wall to allow for the largest lumen area for greater drawing up and lower injection pressure without affecting its robustness. This needle (for personal safety) used in conjunction with the 2ml Acuject syringe and the latest scratch off symbol system provides clear identification if injecting with others. This will substantially reduce the risk of accidental sharing and possible infections.

#### 10 x Acucan Green 21g 1 1/2" Needle

This needle is much larger in diameter and length and is provided to draw the steroid from the vial or ampoule. It is manufactured to the exact same standards as the blue 23g needle so can be used for injecting. The larger lumen of this needle would promote easier injecting of oil based steroids and may be preferable when injecting into the buttock. Slightly warming the oil-based steroid will make it less viscous and easier to inject. Each needle is individually wrapped and sterile.

#### 10 x Acuject Coloured Low Dead Space 2ml Syringe with scratch off identification system

This syringe is individually wrapped and sterile ready for use. It is marked up to 2.5ml to meet the demands of varying users. This syringe (for personal safety) is low dead space, coloured and has the latest scratch off symbol system that provides clear identification of a syringe if injecting with others. This will reduce substantially the risk of accidental sharing and possible infections.

#### 10 x Alcohol Swabs

Each swab is individually wrapped for convenience and contains 70% Isopropyl Alcohol. The swab is used to clean and prepare the area of injection. Please allow time for the area to dry fully before injecting.

