

What is the 2ml Standard Kit?

The standard 2ml pack has been developed to provide equipment of the highest quality that complies with N.I.C.E. PH52 Guidelines. It is designed for groin injecting drug users (IDUs) to promote the best and safest injecting practices. All equipment contained is sterile and for single use only. This pack has been designed for opiate injectors but can also be used by various injectors.. The equipment is designed for intravenous use.

Contents:

This pack will provide enough equipment to last 2 to 3 days.

1 x SAFELOC™ 0.35 needle exchange container

This is for the safe disposal of used injecting equipment. It fits neatly into a pocket or bag and can display bespoke messages on the label. This container is Kite Mark compliant with BS EN ISO 23907:2012 and Compliant to UN3291 - UN certificate UN/3H1/Y1/S/15/GB/6873. This container is the latest innovative product designed specifically for (IDUs) by Orion.

10 x Acucan Orange 25g x 1" Needle

This has been designed specifically for (IDUs). The needle has three bevels of sharpness and is silicone coated. This provides minimum penetration force and skin trauma during insertion. Additionally, the needle has a thin wall to allow for the largest lumen area for greater drawing up without affecting its robustness. This needle (for personal safety) used in conjunction with the 2ml Acuject coloured low dead space syringe and the latest scratch off symbol system provides clear identification if injecting with others. This will substantially reduce the risk of accidental sharing and possible infections.

10 x Acuject Coloured Low Dead Space 2ml Syringe with scratch off identification system

This syringe is individually wrapped and sterile ready for use. It is marked up to 2.5ml to meet the demands of varying users. This syringe (for personal safety) is low dead space, coloured and has the latest scratch off symbol system that provides clear identification of a syringe if injecting with others. This will reduce substantially the risk of accidental sharing and possible infections.

10 x Ascorbic Acid Sachets (Vit C)

This is a pharmaceutical grade powder that has been designed specifically for converting alkaloids such as heroin into an injectable solution. The powders are pre-packed into sachets of 300mg ready for use and are sterile. Ascorbic acid produces a weaker acidic solution than Citric and therefore this reduces the burning affect during injecting.

10 x Orion Sterile Spoons with filter

Orion's spoon is a one-piece spoon manufactured from high quality steel. This allows cooking up without burning to the finger and thumb. It is pillow packed and sterilised for safety and convenience. The pack also holds a small filter designed to allow for safe drawing up whilst retaining minimum solution.

10 x Alcohol Swabs

Each swab is individually wrapped for convenience and contains 70% Isopropyl Alcohol. The swab is used to clean and prepare the area of injection. Please allow time for the area to dry fully before injecting.

